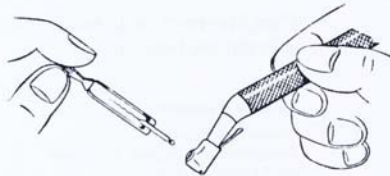
Aseptor- Kassette

mit Klemme

*Aseptor-Cassette
with clamp*

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www.walser-dental.com
E-mail: info@walser-dental.com

**Tens of thousands of dentists in more than 120 countries worldwide work with
WALSER® Matrices and that number increases every day...**

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The Aseptor Cassette

The Aseptor Cassette has been specially designed for holding root canal treatment instruments and for their easy sterilisation. Multiple openings ensure excellent ventilation during sterilisation. After drying, the openings can be closed off by the sliders and guarantee sterile storage.

The contents of the Cassette consist of two punched-out strips which may be folded out, in which receptacles either powered root-canal-treatment instruments can be secured by the specially-designed Clamps, or manual instruments can be placed in suitably-sized holes. The inside of the cover holds two receptacles for cotton-wool rolls and paper inserts.

The Aseptor Clamp has been developed for powered instruments. The Clamps allow easy withdrawal of the instruments from the punched-out receptacles. The instruments can then be placed in an angled powered drill head without being handled at all. The Clamp can be easily removed after locating the instrument by twisting. The miniature head is best suited to this operation.

Note that while filling the Cassette, the instruments are first of all to be placed in the receptacle, and then the Clamp is to be pushed onto the cylindrical shaft as shown in the diagram.

The standard equipment of the Aseptor Cassette consists of one frame for the manual instruments and ten clamps for the powered instruments. It is also possible to equip the Cassette according to need.

Aseptor-Cassette

Length: 7.1 inches
Equipment: 2 frames, 10 clamps or 1 frame, 24 clamps

Aseptor-Cassette Junior

Length: 5.5 inches
Equipment: 1 frame, 10 clamps

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Tens of thousands of dentists in more than 120 countries worldwide work with WALSER® Matrices and that number increases every day...